

Syclone Air/Water Separator Return Procedure

1. Fill out "Generator Section" of waste profile form.
2. Sign and date, then place in the return shipping box.
3. Before shutting off the vacuum:
 - a. Place 1 ½" black pipe cap over both the inlet and outlet pipe
 - b. Insert red plug into hole underneath the air/water separator
4. Place capped system into clear plastic bag and close with twist tie.
5. Flip system upside down and place into 6-gallon bucket.
6. Place plastic lid onto bucket and secure tightly.
7. Take note of the "shipping code" located on the blue lid of bucket.
8. Place bucket into the return shipping box.
9. Use packaging tape (not included) to seal box for shipment.
10. Print UPS label: visit solmetex.com, select "Compliance Center" from header menu, choose "Print Shipping Labels", enter your zip code and account # (found on your picking slip) & print your UPS label
11. Affix prepaid return UPS label on box

